



Fouchet's Reagent

R018

Fouchet's Reagent is used to test for bilirubin and some other derivatives.

Composition**

Ingredients

Trichloroacetic acid	25.00 gm
Ferric chloride,10% aqueous	10.00 ml
Distilled water	100.00 ml

**Formula adjusted, standardized to suit performance parameters

Directions

To about 10 ml urine, add 1 gm barium chloride, mix and filter. Spread out on a filter paper. When it is partly dry, drop a little Fouchet's reagent or yellow nitric acid on the precipitate.

Principle And Interpretation

For testing bilirubin in urine, two types of tests are available namely ,

- 1) oxidation test-where bilirubin is oxidised to green biliverdin and
- 2) diazotization test-where bilirubin is diazotised to a highly coloured compound.

Fouchet's reagent used in Fouchet's test comes under oxidizing reagent or test. Barium chloride precipitates the sulphate radicals present in urine to form precipitate of barium sulphate. If bile pigments are present in urine, they adhere to these molecules. Ferric chloride present in Fouchets reagent then oxidises yellow bilirubin, in the presence of trichloroacetic acid to green biliverdin. Therefore the development of green colour due to the formation of biliverdin indicates the presence of bilirubin (bile) in urine.

Type of specimen

Clinical : Urine

Specimen Collection and Handling

1.For clinical samples follow appropriate techniques for handling specimens as per established guidelines

Warning and Precautions

In Vitro diagnostic use only. Read the label before opening the container. Wear protective gloves/protective clothing/ eye protection/face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Performance and Evaluation

Performance of the product is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Yellow coloured solution

Clarity

Clear without precipitate

Test

To about 10 ml urine, add 1 gm barium chloride, mix and filter, spread out the filter paper. When it is partly dry, drop a little fouchet's reagent or yellow nitric acid on the precipitate

Results

Development of green colour due to formation of biliverdin indicates the presence of bile (bilirubin) in the urine.

Storage and Shelf Life

Store below 30°C in tightly closed container and away from bright light. Use before expiry date on label. On opening, product should be properly stored in dry ventilated area protected from extremes of temperature and sources of ignition. Seal the container tightly after use.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (5,6).

Reference

- 1) Textbook of Medical laboratory Technology; Praful B. Godkar
- 2) Lapage S., Shelton J. and Mitchell T., 1970, Methods in Microbiology', Norris J. and Ribbons D., (Eds.), Vol. 3A, Academic Press, London.
- 3) MacFaddin J. F., 2000, Biochemical Tests for Identification of Medical Bacteria, 3rd Ed., Lippincott, Williams and Wilkins, Baltimore.


Revision : 02 / 2016

IVD

In vitro diagnostic medical device



CE Marking

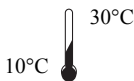

Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,
Swastik Disha Business Park, Via Vadhani
Ind. Est., LBS Marg, Mumbai-400086,
India. www.himedialabs.com

EC REP

CE Partner 4U ,Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.