



# Technical Data

## Dermato Supplement

FD015

An antibiotic supplement, recommended for the selective isolation of pathogenic dermatophytes.

### Composition

Per vial sufficient for 500 ml medium

*Ingredients	Concentration
Cycloheximide	250mg
Chlortetracycline	50mg
Gentamicin	50mg

### Directions:

Rehydrate the contents of 1 vial with 5 ml of 50% v/v acetone. Mix well and aseptically add it to 500 ml of sterile, molten, cooled (45-50°C) D.T.M. Agar Base [M188](#) / D.T.M. Agar Base, Granulated (Dermatophyte Test Agar Base, Granulated) [GM188](#). Mix well and pour into sterile petri plates.

### Storage and Shelf Life

Store at 2-8°C. Use before expiry date on the label

\* Not For Medicinal Use


Revision : 02/2021



In vitro diagnostic medical device



CE Marking

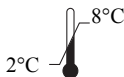

Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,  
23 Vadhani Industrial Estate,  
LBS Marg, Mumbai-86, MS, India



CE Partner 4U ,Esdoornlaan 13, 3951  
DB Maarn The Netherlands,  
[www.cepartner4u.eu](http://www.cepartner4u.eu)

### Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal diagnostic or therapeutic use but for laboratory, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.