



# Technical Data

## Oleic Albumin Supplement

FD020

An enrichment supplement recommended for cultivation of *Mycobacterium tuberculosis*.

### Composition

Per vial sufficient for 200 ml medium

#### Ingredients

Alkalinized oleic acid

Albumin fraction V

#### Concentration

0.050%

5%

### Directions:

Warm up the refrigerated supplement to room temperature. Shake well and aseptically add the entire content (20 ml) of 1 vial of sterile Oleic Albumin Supplement to 180 ml of sterile, molten, cooled (45-50°C) Dubos Oleic Agar Base [M179](#) / Dubos Oleic HiVeg™ Agar Base [MV179](#) / Dubos Oleic Broth Base [M839](#) / Dubos Oleic HiVeg™ Broth Base [MV839](#) along with 5000 to 10000 units of penicillin. Mix well.

### Storage and Shelf Life

Store at 2-8°C. Use before expiry date on the label.


Revision : 02/2021



In vitro diagnostic medical device



CE Marking

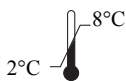

Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,  
23 Vadhani Industrial Estate,  
LBS Marg, Mumbai-86, MS, India



CE Partner 4U ,Esdoornlaan 13, 3951  
DB Maarn The Netherlands,  
[www.cepartner4u.eu](http://www.cepartner4u.eu)

#### Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal diagnostic or therapeutic use but for laboratory, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.