



BHI Broth- Supplemented w/ 0.05% SPS

LQ004A

Intended Use

Recommended as a qualitative test for detection of microorganisms in blood. *Sterile, in glass bottles.*

Composition**

Ingredients	Gms / Litre
HM infusion powder #	12.500
BHI powder	5.000
Proteose peptone	10.000
Dextrose (Glucose)	2.000
Sodium chloride	5.000
Disodium hydrogen phosphate	2.500
SPS	0.500
Final pH (at 25°C)	7.4±0.2

**Formula adjusted, standardized to suit performance parameters

Equivalent to Calf brain infusion from

Directions

Label the ready to use blood culture bottle. Remove the Aluminium foil cap. Disinfect the part of the rubber stopper which is now exposed. Draw patient's blood with the sterile or disposable needle and syringe as explained in specimen collection and disposable column. Transfer the blood sample immediately into the culture bottle by puncturing the rubber stopper with the needle and injecting the blood. Venting: Use sterile venting needle (LA038). Keep the bottle in an upright position preferably in a biological safety cabinet, place an alcohol swab over the rubber stopper and insert the venting needle with filter through it. Insertion and withdrawal of the needle should be done in a straight line. discard the needle and mix the contents by gently inverting the bottle 2-3 times. Do Not vent the bottle for anaerobic cultures. Incubate at 35±2°C for 18-24 hours and further for seven days.

Principle And Interpretation

BHI Medium is useful for cultivating a wide variety of microorganisms since it is a highly nutritive medium. It is also used to prepare the inocula for antimicrobial susceptibility testing. BHI Broth is a modification of the original formulation of Rosenow, where he added pieces of brain tissues to dextrose broth (10). BHI Broth is also the preferred medium for anaerobic bacteria, yeasts and moulds (8,1). This medium is nutritious and well buffered to support the growth of wide variety of organisms (8,10,4). With the addition of 10% defibrinated sheep blood, it is useful for isolation and cultivation of *Histoplasma capsulatum* (5) and other fungi. For selective isolation of fungi, addition of gentamicin and/or chloramphenicol is recommended (9).

Proteose peptone, HM infusion powder and BHI powder serve as sources of carbon, nitrogen, essential growth factors, amino acids and vitamins. Dextrose serves as a source of energy. Disodium phosphate helps in maintaining the buffering action of the medium whereas sodium chloride maintains the osmotic equilibrium of the medium. SPS acts as an anticoagulant and as an inhibitor of the bacteriostatic and bactericidal effects of blood cells and plasma factors (4,5).

Type of specimen

Clinical samples : Blood

Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (6,7). After use, contaminated materials must be sterilized by autoclaving before discarding.

Limitations

1. Further biochemical and serological tests must be carried out for complete identification.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Sterile clear BHI Broth Supplemented w/ 0.05% SPS in glass bottle.

Colour

Light amber coloured clear solution

Quantity of Medium

70ml of medium in glass bottle, (For Adult use)

Reaction

7.20- 7.60

Sterility test

Passes release criteria

Cultural response

Cultural characteristics was observed after incubation at 35-37°C for 24-48 hours.

Organism	Inoculum (CFU)	Growth
<i>Neisseria meningitidis</i> ATCC 13090	50-100	good-luxuriant
<i>Streptococcus pneumoniae</i> ATCC 6303	50-100	good-luxuriant
<i>Streptococcus pyogenes</i> ATCC 19615	50-100	good-luxuriant
<i>Candida albicans</i> ATCC 10231 (00054*)	50-100	good-luxuriant
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 25923 (00034*)	50-100	good-luxuriant
<i>Enterococcus faecalis</i> ATCC 29212 (00087*)	50-100	good-luxuriant
<i>Escherichia coli</i> ATCC 25922 (00013*)	50-100	good-luxuriant
<i>Shigella flexneri</i> ATCC 12022 (00126*)	50-100	good-luxuriant

Key : (*) Corresponding WDCM numbers.

Storage and Shelf Life

Store between 15-25°C. Use before expiry date on the label.

Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (6,7).

Reference

1. Atlas R. M., 1993, Handbook of Microbiological Media, 147-153, CRC Press, Boca Raton, FL.
2. Belding, M. E., and S. J. Klebanoff. 1972. Effect of sodium polyanethole sulfonate on antimicrobial systems in blood. Appl. Microbiol. 24:691-698.
3. Eng, J. 1975. Effect of sodium polyanethol sulfonate in blood cultures. J. Clin. Microbiol. 1:119-123.
4. Conant N. F., 1950, Diagnostic Procedures and Reagents, 3rd Ed., APHA Inc., New York
5. Howard B., Keiser J. F., Weissfeld A. et al, 1994, Clinical and Pathogenic Microbiology, 2nd Ed., Mosby Co.
6. Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2nd Edition.
7. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.
8. MacFaddin J. F., 1985, Media for Isolation-Cultivation-Identification-Maintenance of Medical Bacteria, Vol. I, Williams and Wilkins, Baltimore.
9. Murray P. R., Baron J. H., Pfaller M. A., Jorgensen J. H. and Tenover F. C., (Eds.), 2003, Manual of Clinical Microbiology, 8th Ed., American Society for Microbiology, Washington, D.C.
10. Rosenow, 1919, J. Dental Research, 1:205.
11. Roseburg T. et al, 1944, J. Inf. Dis., 74:131


Revision : 01 / 2019



In vitro diagnostic medical device



CE Marking

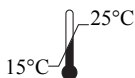

Storage temperature



Do not use if package is damaged

HiMedia Laboratories Pvt. Limited,
23 Vadhani Industrial Estate,
LBS Marg, Mumbai-86, MS, IndiaCE Partner 4U, Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.