
HiViral™ Transport Kit

With 3ml Viral Transport Medium in 10 - 15ml screw cap tube and two sterile individually packed synthetic fiber swabs with breakpoint

Product Code: MS2760S1

Intended Use: For collection and transportation of samples containing Viruses Chlamydiae, Mycoplasma and Ureaplasma from the collection site to the laboratory

Introduction:

HiMedia's HiViral™ Transport Kit is a specially designed transport system to collect and transport viruses in an active form to the laboratory for isolation. It is designed to maintain the viability and the virulence of the viral sample. It contains 2.5ml to 3ml viral transport medium in self-standing or conical bottom tubes.

HiViral™ Transport Medium is made of Hanks Balanced Salt Solution and contains a protective protein, antibiotics to control microbial and fungal contamination and buffers to control the pH. Phenol red is used as a pH indicator. The medium also contains a cryoprotectant which helps in preserving the viruses if specimens are frozen for prolonged storage.

The nasopharyngeal swab has a short perpendicular ultra-flexible plastic shaft that is designed for better patient comfort. It is efficient in collection and release of particulate matter. It yields significantly more sample which helps in maximizing the sensitivity of serological and molecular detection assays. The oropharyngeal swab is used for collecting throat samples. Both the swabs have breakpoint.

Kit Contents:

1. HiViral™ Transport Medium
2. Sterile nasopharyngeal swab with breakpoint
3. Sterile oropharyngeal swab with breakpoint

Procedure:

A. Collection of Samples

For a complete diagnostic analysis of viral diseases, it is important that the infectivity of the viruses is preserved after sample collection. The infectivity of viruses' decreases over time and the decay rate is generally a function of temperature. Stability of samples is enhanced by cooling therefore samples should be kept at 2-8°C. The probability of a successful isolation is more if the samples are processed immediately after collection and the viral load in the sample is more. Viral load is maximum if the samples are collected immediately after the onset of clinical symptoms and before the administration of antiviral medications.

B. Directions:

1. Cut open the pouch to remove the swab.
2. Specimen can be collected with the swab in the following manner.

Nasopharyngeal swab

Insert dry swab in to nostril and back to the nasopharynx. Leave in place for a few seconds. Slowly withdraw the swab with a rotating motion.

Oropharyngeal swab

Ask patient to open his/her mouth. Swab the back of the throat near the tonsils thoroughly.

3. Break the swab near the break point and insert swab into the tube containing viral transport medium and close the cap tightly.
4. Label the sample correctly with the name of the patient and time and date of collection.
5. Transport the samples immediately to the laboratory for processing.

Transportation of the Samples:

Samples should be transported to the laboratory as soon as possible.

Samples can be refrigerated at 2-8°C after collection or can be transported at 2-8°C on wet ice within 48 hours. If a long delay is expected in transit and processing, samples should be transported on dry ice and should be frozen at -70°C.

Precautions:

1. Isolation of viruses will largely depend on proper specimen collection, timing of sample collection and processing of samples.
2. Specimen collection should be done in the acute phase of illness.
3. Do not use the product if (1) there is change in the color of the medium,(2) there is evidence of leakage,(3) there are other signs of deterioration.
4. To maintain infectivity of viruses it is important that temperature be properly maintained for sample collection to processing.
5. Avoid repeated freeze-thaw of collected samples.
6. It is recommended to refer to standard procedures and published protocols for sample collection and processing.

Quality control:

Appearance

Orange coloured clear solution

pH at 25°C

7.3 ± 0.3

Osmolality in mOsm/Kg H₂O

500.00 - 600.00

Sterility

No bacterial or fungal growth is observed after 14 days of incubation as per USP specification.

Method of sterilization:

1. Media: Filter sterilization using 0.22 micron filter
2. Swabs: ETO sterilization

Storage and shelf life:

Store at 15-30°C.

Use before expiry date given on the product label.



In vitro diagnostic medical device



CE Marking



ICMR Approved

Revision 0/2020

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ Publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal diagnostic or therapeutic use but for laboratory, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.